## ACID CONCENTRATE FOR HEMODIALYSIS

### INSTRUCTIONS FOR USE:

#### IMPORTANT:

Keep at a temperature no lower than 4°C.

Single use, pyrogen-free product.

#### INDICATIONS:

For the treatment of acute or chronic kidney failure.

For additional information, contact the manufacturer.

#### INSTRUCTIONS:

To be used in hemodialysis machines for powder sodium bicarbonate dilution 1:44.

Dilution based on the following proportion: See the label of the product. B is the bicarbonate concentrate and H<sub>2</sub>O is deionized water complying with regulations for quality of reagents.

To be used with the machine acid connector, following the instructions for use of the hemodialysis machine manufacturer.

#### WARNINGS:

To be used under the physician's orders.

Make sure the composition corresponds with the physician's prescription.

To be used by qualified personnel.

Do not use if package is damaged.

The concentrated solution must be diluted immediately before use. Any quantity of unused concentrated solution must be discarded. On the contrary, neither the properties of the product can be guaranteed nor the full mixture composition; therefore, the best treatment for the patient cannot be assured.

See below for table indicating hemodialysis minutes for standard dialysis bath flows according to the container volume.

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