ACID CONCENTRATE FOR HEMODIALYSIS INSTRUCTIONS FOR USE:

IMPORTANT:

Keep at a temperature no lower than 4°C.

Product-apigeno.

INDICATIONS:

For the treatment of acute or chronic kidney failure. For additional information, contact the manufacturer.

INSTRUCTIONS:

To be used in hemodialysis machines for powder bicarbonate dilution 1:44.

To be used with the machine acid connector, following the instructions for use of the hemodialysis machine manufacturer. Dilution based on the following proportion: See the label of the product, B is the bicarbonate concentrate and H₂O is deionized water complying with local applicable quality requirements.

WARNINGS:

To be used under the physician's order.

Make sure the composition corresponds with the physician's prescription.

To be used by qualified personnel.

Do not use if package is damaged.

The concentrate solution must be diluted immediately before use. Must be used for medical treatments.

Once open, the product must be used within the following 24 hours. Close the container after every use.

See below for table indicating hemodialysis minutes for standard dialysis bath flows according to the container volume.

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