NIPROTANK-D CITRASATE™

341-C

EN: Instructions for use

Important:

Keep at a temperature no less than 4°C, Pyrogen-free product.

Indications:

For use in the preparation of dialysis fluid used in hemodialysis and related therapies, for the treatment of acute or chronic renal failure.

Instructions:

To be used in hemodialysis machines for powder bicarbonate dilution 1:44. To be connected to the acid concentrate connector of the central concentrate supply system. The acid concentrate from the central concentrate supply system must be used with a dialysis machine, following the instructions of the manufacturer of the machine. Dilution is based on the following proportion: See the label of the product. B is the bicarbonate concentrate and H₂O is deionized water, complying with applicable local quality requirements.

Warnings:

To be used under the physician's order. Make sure the composition corresponds with the physician's prescription. To be used by qualified personnel. Do not use if package is damaged. The concentrated solution must be diluted immediately before use. Once opened, the product must be used continuously. Do not use after disconnection from the central concentrate supply system. Do not use after the expiry date stated on the package. Citrate, a component of this product, may reduce the need for heparin and may lower plasma Ca and Mg concentrations. Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is based. Dispose of the product according to local and national regulations.

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